

LZI Norbuprenorphine Controls

IVD For In Vitro Diagnostic Use Only



Lin-Zhi International, Inc.

REF	Description	Quantity
A68824	Norbuprenorphine 7 ng/mL Level 1 Control	1 x 5 mL
A68825	Norbuprenorphine 13 ng/mL Level 2 Control	1 x 5 mL

Intended Use


The Lin-Zhi International, Inc. (LZI) Norbuprenorphine Controls are for use as assayed quality control materials to monitor the precision of the LZI Buprenorphine Enzyme Immunoassay with Beckman Coulter® Synchron DxC automated clinical system analyzers (1).

Description of the Controls

The LZI Norbuprenorphine Controls are human urine-based liquids and ready-to-use. The constituent is a processed drug-free human urine matrix containing buffers, stabilizers, and less than 0.1 % of sodium azide. The controls are prepared by spiking known concentrations of norbuprenorphine into the drug-free matrix.

*Actual concentrations of these calibrators are within $\pm 10\%$ of the target value as determined by GC/MS or LC/MS. Values are provided only as guidelines and laboratories should determine the ranges based on their own test system and tolerance (2).

Precautions and Warning

- The LZI Norbuprenorphine Controls are for in vitro diagnostic use only. Harmful if swallowed.
- The controls contain sodium azide, which may react with lead or copper plumbing to form potentially explosive metal azide. When disposing such liquids always flush with a large volume of water to prevent azide build-up (3).
- The controls are prepared from non-sterile human urine. They are not tested by licensed reagents for the presence of antibodies to human immunodeficiency viruses, the hepatitis antigens, and/or anti-hepatitis antibodies. They should be handled as potentially infectious. Always apply good laboratory practice to avoid any skin contact or ingestion.
- Do not use the controls beyond their expiration dates.
-  For USA: Caution: Federal law restricts this device to sale by or on the order of a physician.

Preparation and Storage

The controls are provided ready-to-use. No reconstitution is required. Label the cap before removal to identify it with the original bottle. The controls should be stored refrigerated at 2-8°C when not in use.

Stability

When stored refrigerated at 2-8°C, the controls are stable either opened-recapped or unopened until the expiration date printed on the vial label. Store controls tightly capped when not in use. Control solution dispensed in the sample cups and left on board the clinical analyzer should be discarded after use.

Procedure and Results

Both levels of controls (7.5 ng/mL and 12.5 ng/mL for the 10 ng/mL cutoff) should be run daily to ensure proper assay performance. Additionally, run the controls with each new calibration and after specific maintenance or troubleshooting procedures are performed. For interpretation of results, refer to the appropriate LZI Buprenorphine Enzyme Immunoassay for Beckman Coulter Synchron Systems (Ref # A53684) package insert (1).

Limitations

The LZI Norbuprenorphine Controls are for use with the LZI Buprenorphine Enzyme Immunoassay for Beckman Coulter Synchron Systems (Ref # A53684) to detect norbuprenorphine in human urine only. Quality control materials should be used in accordance with local, state, and/or federal regulations or accreditation requirements.

Bibliography

1. LZI Buprenorphine Enzyme Immunoassay for Beckman Coulter® Synchron Systems (Ref # A53684) package insert.
2. Guidance for Industry and FDA Staff, Assayed and Unassayed Quality Control Material. U.S. Department of Health and Human Services. FDA, Document issued on June 7, 2007.
3. Sodium Azide. National Institute for Occupational Safety (NIOSH) Pocket Guide to Chemical Hazards. Third Printing, September 2007. Available online at: <https://www.cdc.gov/niosh/npg/default.html>


Notice: Adulteration of reagents, use of instruments without appropriate capabilities, or other failure to follow instructions as set forth in this labeling can affect performance characteristics, and stated or implied label claims.

Additions, deletions, or changes are indicated by a change bar in the margin.



Manufacturer:

Lin-Zhi International, Inc.
2945 Oakmead Village Court
Santa Clara, CA 95051
USA
Tel: (408) 970-8811
Fax: (408) 970-9030
www.lin-zhi.com

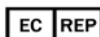

Authorized European Rep. within the EU:

CEpartner4U
Esdoornlaan 13
3951 DB Maarn
The Netherlands
www.cepartner4u.eu

