

REF 0530c (100/37.5 mL R<sub>1</sub>/R<sub>2</sub> Kit)  
0531c (1000/375 mL R<sub>1</sub>/R<sub>2</sub> Kit)**Lin-Zhi International, Inc.****Intended Use**

The LZI Ethyl Glucuronide III Enzyme Immunoassay is intended for the qualitative and semi-quantitative determination of ethyl glucuronide in human urine at the cutoff value of 500 ng/mL when calibrated against ethyl glucuronide. The assay is designed for use with a number of automated clinical chemistry analyzers.

The semi-quantitative mode is for purposes of (1) enabling laboratories to determine an appropriate dilution of the specimen for confirmation by a confirmatory method such as gas or liquid chromatography/mass spectrometry (GC/MS or LC/MS) or (2) permitting laboratories to establish quality control procedures.

**The assay provides only a preliminary analytical result. A more specific alternative analytical chemistry method must be used in order to obtain a confirmed analytical result. Gas or Liquid Chromatography/Mass Spectrometry (GC/MS or LC/MS) are the preferred confirmatory methods (1, 2). Clinical consideration and professional judgment should be exercised with any drug of abuse test result, particularly when the preliminary test result is positive.**

**Summary and Explanation of Test**

Ethyl glucuronide (EtG) is an ethanol metabolite that is formed from the conjugation of ethanol to glucuronic acid (3, 4). Although EtG is a minor metabolite of ethanol, composing less than 0.05 % of the ingested ethanol dose (4, 5), compared to ethanol alone, its longer detection time has made it an increasingly popular biomarker for alcohol consumption (6-9). Due to rapid metabolism and excretion, the time frame for ethanol detection in urine is normally less than 12 hours (10). Depending on the dosage of ethanol ingested, EtG has a detection window of up to four days following elimination of ethanol from the body (4, 6, 7, 11-13).

EtG can be used to screen for recent alcohol intake as well as chronic alcoholism due to its non-volatile and water-soluble properties (14-18).

**Assay Principle**

The LZI Ethyl Glucuronide III Enzyme Immunoassay is a homogeneous enzyme immunoassay ready-to-use liquid reagent. The assay is based on competition between ethyl glucuronide in the sample and ethyl glucuronide-labeled with the enzyme glucose-6-phosphate dehydrogenase (G6PDH) for a fixed amount of antibody in the reagent (19). Enzyme activity decreases upon binding to the antibody, and the ethyl glucuronide concentration in the sample is measured in terms of enzyme activity. In the absence of ethyl glucuronide in the sample, ethyl glucuronide-labeled G6PDH conjugate is bound to antibody, and the enzyme activity is inhibited. On the other hand, when ethyl glucuronide is present in the sample, antibody would bind to free drug; the unbound ethyl glucuronide-labeled G6PDH then exhibits its maximal enzyme activity. Active enzyme converts nicotinamide adenine dinucleotide (NAD) to NADH, resulting in an absorbance change that can be measured spectrophotometrically at 340 nm.

**Reagents Provided**

**Antibody/Substrate Reagent (R<sub>1</sub>):** Contains mouse monoclonal anti-ethyl glucuronide antibody, glucose-6-phosphate (G6P), nicotinamide adenine dinucleotide (NAD), stabilizers, and sodium azide (0.09 %) as a preservative.

**Enzyme-drug Conjugate Reagent (R<sub>2</sub>):** Contains glucose-6-phosphate dehydrogenase (G6PDH) labeled with ethyl glucuronide in buffer with sodium azide (0.09 %) as a preservative.

**Calibrators and Controls\***

\*Calibrators and Controls are sold separately and contain negative human urine with sodium azide as a preservative.

ETHYL GLUCURONIDE (500) Calibrators	REF
Low Calibrator	0001
Low Calibrator: Contains 200 ng/mL ethyl glucuronide	0542c
Cutoff Calibrator: Contains 500 ng/mL ethyl glucuronide	0543c
Intermediate Calibrator: Contains 1000 ng/mL ethyl glucuronide	0544c
High Calibrator: Contains 2000 ng/mL ethyl glucuronide	0545c

ETHYL GLUCURONIDE (500) Controls	REF
Level 1 Control: Contains 375 ng/mL ethyl glucuronide	0547c
Level 2 Control: Contains 625 ng/mL ethyl glucuronide	0548c

**Precautions and Warning**

- This test is for in vitro diagnostic use only. Harmful if swallowed.
- Reagent contains sodium azide as a preservative, which may form explosive compounds in metal drain lines. When disposing such reagents or wastes, always flush with a large volume of water to prevent azide build-up. See National Institute for Occupational Safety and Health Bulletin: Explosive Azide Hazards (20).
- Do not use the reagents beyond their expiration dates.

**Reagent Preparation and Storage**

The reagents are ready-to-use. No reagent preparation is required. All assay components should be refrigerated at 2-8°C when not in use.

**Specimen Collection and Handling**

Urine samples may be collected in plastic or glass containers. Use fresh urine specimens for the test. If the sample cannot be analyzed immediately, it may be refrigerated at 2-8°C for at least seven days (21). For longer storage, keep sample frozen at -20°C, before using the sample, completely mix (vortex), thaw additionally overnight at 2-8°C, then test. Studies have shown ethyl glucuronide analytes in urine are stable frozen at -20°C up to three months. Samples should be equilibrated to room temperature (18-25°C) for testing. Samples with high turbidity should be centrifuged before analysis.

Adulteration may cause erroneous results. If sample adulteration is suspected, obtain a new sample and both samples should be forwarded to a laboratory for testing.

Handle all urine specimens as if they are potentially infectious.

**Instrument**

Clinical chemistry analyzers capable of maintaining a constant temperature, pipetting sample, mixing reagents, measuring enzyme rates at 340 nm and timing the reaction accurately can be used to perform this homogeneous immunoassay.

Performance characteristics presented in this package insert have been validated on the Beckman Coulter® AU480. If other instruments are used, performance will need to be validated by the laboratory (22, 23).

**Assay Procedure**

Typical assay parameters used for the Beckman Coulter AU480 analyzer include a 15 µL sample, 80 µL of antibody reagent (R<sub>1</sub>), 30 µL of enzyme conjugate reagent (R<sub>2</sub>), 10 µL dilution following addition of R<sub>2</sub> in 37°C incubation temperature, 15-21 reading points, FIXED method, and 340 nm primary wavelength.

For qualitative analysis use the 500 ng/mL as the cutoff calibrator. For semi-quantitative analysis, use all five calibrators. Recalibration should be performed after reagent bottle change or a change in calibrators or reagent lot. Two levels of controls are also available for monitoring the cutoff level: 375 ng/mL and 625 ng/mL.

**Calibration and Quality Control**

Good laboratory practices recommend the use of at least two levels of control specimens (one positive and one negative control near the cutoff) to ensure proper assay performance. Controls should be run with each new calibration and after specific maintenance or troubleshooting procedures as detailed in the instrument system manual. Each laboratory should establish its own control frequency. If any trends or sudden change in control value are observed, review all operating parameters, or contact LZI technical support for further assistance. Laboratories should comply with all federal, state, and local laws, as well as all guidelines and regulations.

**Results**

**Note:** A preliminary positive test result does not necessarily mean a person took a specific drug and a negative test result does not necessarily mean a person did not take a specific drug. There are a number of factors that influence the reliability of drug tests.

**Qualitative:** The cutoff calibrator, which contains 500 ng/mL of ethyl glucuronide, is used as a reference for distinguishing a preliminary positive from negative samples. A sample with a change in absorbance (ΔmAU) equal to or greater than that obtained with the cutoff calibrator is considered a preliminary positive. A sample with a change in absorbance (ΔmAU) lower than that obtained with the cutoff calibrator is considered negative.

**Semi-Quantitative:** The semi-quantitative mode is for purposes of (1) enabling laboratories to determine an appropriate dilution of the specimen for verification by a confirmatory method such as GC/MS, LC/MS or (2) permitting laboratories to establish quality control procedures. When an approximation of concentration is required, a calibration curve can be established with five calibrators. The concentration of ethyl glucuronide in the sample may then be estimated from the calibration curve.

### Limitations

- Use of hand-sanitizers with isopropanol or n-propanol should be avoided for 12 hours prior to collection of the sample.
- Ingestion of medications and foods with alcohol should be avoided for 12 hours prior to collection of the sample.
- Boric Acid at 1% w/v and Citric Acid (pH 3) may cause false negative results. Boric Acid and Citric Acid are not recommended as a preservative for urine.
- A preliminary positive result from this assay indicates only the presence of ethyl glucuronide and does not necessarily correlate with the extent of physiological and psychological effects (e.g., intoxication). This test is not intended for quantifying the individual analytes in samples.
- A negative result does not necessarily mean a person did not abuse drugs.
- Care should be taken when reporting results, as numerous factors (e.g., fluid intake, endogenous or exogenous interferents) may influence the urine test result.
- Preliminary positive results should be confirmed by other affirmative, analytical methods (e.g., chromatography), preferably GC/MS or LC/MS.
- The test is designed for use with human urine only.
- The test is not for therapeutic drug monitoring.

### Typical Performance Characteristics

The results shown below were performed with a single Beckman Coulter AU480 automated chemistry analyzer.

#### Precision:

**Semi-quantitative analysis:** The following concentrations were determined with reference curves from five calibrators.

500 ng/mL Cutoff		Within Run (N=22)		Run-to-Run (N=88)	
Concentration	% of Cutoff	# Samples	EIA Result	# Samples	EIA Result
0 ng/mL	0.0 %	22	22 Neg	88	88 Neg
125 ng/mL	25.0 %	22	22 Neg	88	88 Neg
250 ng/mL	50.0 %	22	22 Neg	88	88 Neg
375 ng/mL	75.0 %	22	22 Neg	88	88 Neg
500 ng/mL	100.0 %	22	22 Neg/ 0 Pos	88	79 Neg/ 9 Pos
625 ng/mL	125.0 %	22	22 Pos	88	88 Pos
750 ng/mL	150.0 %	22	22 Pos	88	88 Pos
875 ng/mL	175.0 %	22	22 Pos	88	88 Pos
1000 ng/mL	200.0 %	22	22 Pos	88	88 Pos

**Qualitative analysis:** The following concentrations were evaluated. Typical qualitative results (measured by ΔOD, mAU) are as follows:

500 ng/mL Cutoff		Within Run (N=22)		Run-to-Run (N=88)	
Concentration	% of Cutoff	# Samples	EIA Result	# Samples	EIA Result
0 ng/mL	0.0 %	22	22 Neg	88	88 Neg
125 ng/mL	25.0 %	22	22 Neg	88	88 Neg
250 ng/mL	50.0 %	22	22 Neg	88	88 Neg
375 ng/mL	75.0 %	22	22 Neg	88	88 Neg
500 ng/mL	100.0 %	22	21 Neg/ 1 Pos	88	77 Neg/ 11 Pos
625 ng/mL	125.0 %	22	22 Pos	88	88 Pos
750 ng/mL	150.0 %	22	22 Pos	88	88 Pos
875 ng/mL	175.0 %	22	22 Pos	88	88 Pos
1000 ng/mL	200.0 %	22	22 Pos	88	88 Pos

**Accuracy:** Three hundred and twenty-six (326) unaltered clinical urine specimens were tested with the LZI Ethyl Glucuronide III Enzyme Immunoassay and confirmed by LC/MS. Specimens having an ethyl glucuronide concentration greater than 500 ng/mL by LC/MS are defined as positive, and specimens with an ethyl glucuronide concentration below 500 ng/mL by LC/MS are defined as negative in the table below. Near cutoff samples are defined as ±50 % of the cutoff value. Adjusted LC/MS values have been corrected for cross-reactivity. The correlation results are summarized as follows:

#### Semi-Quantitative Accuracy Study:

500 ng/mL Cutoff	Neg	< 50 % of the cutoff	Near Cutoff Neg	Near Cutoff Pos	High Pos	%Agreement
Positive	0	0	3*	19	102	96.0 %
Negative	28	99	70	4*	1***	98.5 %

The following table summarizes the result for the discordant samples:

500 ng/mL Cutoff	LC/MS	LC/MS Pos/Neg Result	LZI EIA Pos/Neg Result
192*	455	-	512.2 +
196*	476	-	560.4 +
198*	478	-	596.4 +
202**	521	+	440.5 -
204**	544	+	391.8 -
210**	605	+	435.2 -
211**	621	+	475.7 -
238***	975	+	462.4 -

#### Qualitative Accuracy Study:

500 ng/mL Cutoff	Neg	< 50 % of the cutoff	Near Cutoff Neg	Near Cutoff Pos	High Pos	%Agreement
Positive	0	0	3*	18	102	95.2 %
Negative	28	99	70	5*	1***	98.5 %

The following table summarizes the result for the discordant samples:

500 ng/mL Cutoff	LC/MS	Pos/Neg Result	LZI EIA	Pos/Neg Result	Cutoff Value (mAU)
192*	455	-	217.3	+	210.6
196*	476	-	224.5	+	210.6
198*	478	-	238.8	+	210.6
202**	521	+	182.4	-	210.6
203**	531	+	202.0	-	208.2
204**	544	+	166.7	-	197.7
210**	605	+	188.7	-	208.3
211**	621	+	203.7	-	208.3
238***	975	+	194.8	-	210.6

**Analytical Recovery:** To demonstrate linearity for purposes of sample dilution and quality control (see semi-quantitative results section) of the entire assay range, pooled negative human urine spiked with ethyl glucuronide at 2000 ng/mL was serially diluted. Each sample was run in 10 replicates and the average was used to determine percent recovery compared to the expected target value.

Samples from the linear range of the assay were tested with recovery ranging from 86.4% to 114.1%.

Target Concentration (ng/mL)	Determined (ng/mL)	% Recovery
2000	2119.3	106.0%
1800	1929.1	107.2%
1600	1786.2	111.6%
1400	1597.1	114.1%
1200	1269.3	105.8%
1000	984.4	98.4%
800	796.3	99.5%
600	571.3	95.2%
400	366.6	91.7%
200	172.9	86.4%
100	94.4	94.4%
0	0.6	N/A

**Specificity:** Various potentially interfering substances were tested for cross-reactivity with the assay. Test compounds were spiked into pooled negative human urine to various concentrations and evaluated against the cutoff calibrator.

The following table lists the concentration of each test compound that gave a response approximately equivalent to that of the cutoff calibrator (as positive) or the maximal concentration of the compound tested that gave a response below the response of the cutoff calibrator (as negative). Compounds tested at high concentration with results below the cutoff value were listed as Not Detected (ND).

#### Structurally Related Compounds:

Compound	Target [ ] (ng/mL)	% Cross-Reactivity
Ethyl Glucuronide	500	100.5%
Acetaldehyde	1,000,000	0.0%
Ethyl Sulfate	1,000,000	0.0%
Butanol	1,000,000	0.0%
2-Chloroethanol	1,000,000	0.0%
Ethanol	1,000,000	0.0%
Ethylene Glycol	1,000,000	0.0%
Isopropanol	1,000,000	0.0%
Methanol	1,000,000	0.0%
n-Propanol	1,000,000	0.0%
Alprazolam Glucuronide	100,000	0.0%
Acetaminophen-D-glucuronide	100,000	0.0%
Buprenorphine- 3β-D-glucuronide	10,000	0.8%

**Structurally Related Compounds, continued:**

Compound	Target [ ] (ng/mL)	% Cross-Reactivity
Butyl-β-D-glucopyranoside	100,000	0.0%
d-Glucose	3,000,000	0.0%
Ethyl α-D-glucopyranoside	100,000	0.0%
Ethyl β-D-glucopyranoside	229,000	0.3%
Glucuronic Acid	1,000,000	0.0%
Isopropyl-β-D-glucuronide	18,050	2.7%
Methyl Glucuronide	16,000	3.2%
Morphine-3-glucuronide	100,000	0.0%
Morphine-6-glucuronide	100,000	0.0%
4-Nitrophenyl-β-D-glucuronide	100,000	0.0%
Norbuprenorphine-glucuronide	10,000	0.6%
n-propyl-β-D-glucuronide	5390	9.5%
Oxazepam Glucuronide	10,000	0.9%
Phenyl-β-D-glucuronide	100,000	0.0%
Termazepam Glucuronide	10,000	0.8%
Trichloroethyl-β-D-glucuronide	100,000	0.0%

**Structurally Unrelated Compounds:**

Compound	Spiked [ ] (ng/mL)	Spiked Ethyl Glucuronide Concentration (ng/mL)		
		0 ng/mL	375 ng/mL Control	625 ng/mL Control
Acetaminophen	100,000	ND	Neg	Pos
6-Acetylmorphine	10,000	ND	Neg	Pos
Acetylsalicylic Acid	100,000	ND	Neg	Pos
Amitriptyline	100,000	ND	Neg	Pos
Amlodipine Besylate	100,000	ND	Neg	Pos
Amoxicillin	100,000	ND	Neg	Pos
d-Amphetamine	100,000	ND	Neg	Pos
Atorvastatin	20,000	ND	Neg	Pos
Benzoyllecgonine	100,000	ND	Neg	Pos
Buprenorphine	15,000	ND	Neg	Pos
Bupropion	100,000	ND	Neg	Pos
Caffeine	100,000	ND	Neg	Pos
Carbamazepine	100,000	ND	Neg	Pos
Cetirizine	100,000	ND	Neg	Pos
Chlorpheniramine	100,000	ND	Neg	Pos
Chlorpromazine	100,000	ND	Neg	Pos
Clomipramine	100,000	ND	Neg	Pos
Codeine	100,000	ND	Neg	Pos
Desipramine	100,000	ND	Neg	Pos
Diphenhydramine	100,000	ND	Neg	Pos
Duloxetine	100,000	ND	Neg	Pos
Fentanyl (citrate)	10,000	ND	Neg	Pos
Fluoxetine	100,000	ND	Neg	Pos
Fluphenazine	100,000	ND	Neg	Pos
Gabapentin	100,000	ND	Neg	Pos
Hydrocodone	100,000	ND	Neg	Pos
Hydromorphone	100,000	ND	Neg	Pos
Ibuprofen	100,000	ND	Neg	Pos
Imipramine	100,000	ND	Neg	Pos
Lisinopril	100,000	ND	Neg	Pos
Losartan	10,000	ND	Neg	Pos
Loratidine	100,000	ND	Neg	Pos
MDA (3,4-methylenedioxyamphetamine)	100,000	ND	Neg	Pos
MDEA	100,000	ND	Neg	Pos
MDMA (3,4-methylenedioxy-methamphetamine)	100,000	ND	Neg	Pos
Meperidine	100,000	ND	Neg	Pos
Metformin	100,000	ND	Neg	Pos
Metoprolol	100,000	ND	Neg	Pos
Metadone	100,000	ND	Neg	Pos
d-Methamphetamine	100,000	ND	Neg	Pos
Morphine	100,000	ND	Neg	Pos
Nicotine	100,000	ND	Neg	Pos
Nortriptyline	100,000	ND	Neg	Pos
Omeprazole	100,000	ND	Neg	Pos
Oxazepam	100,000	ND	Neg	Pos
Oxycodone	100,000	ND	Neg	Pos
Oxymorphone	100,000	ND	Neg	Pos
Phenobarbital	100,000	ND	Neg	Pos
(1S,2S)-(+)-Pseudoephedrine	100,000	ND	Neg	Pos
Quetiapine	100,000	ND	Neg	Pos
Ramitidine	100,000	ND	Neg	Pos
Salbutamol (Albuterol)	100,000	ND	Neg	Pos
Setraline	100,000	ND	Neg	Pos
THC-COOH (11-Nor-Δ <sup>9</sup> -THC-9-carboxylic acid)	1000	ND	Neg	Pos

**Structurally Unrelated Compounds, continued:**

Compound	Spiked [ ] (ng/mL)	Spiked Ethyl Glucuronide Concentration (ng/mL)		
		0 ng/mL	375 ng/mL Control	625 ng/mL Control
l-Thyroxine	10,000	ND	Neg	Pos
Tramadol	100,000	ND	Neg	Pos
Zolpidem	10,000	ND	Neg	Pos

*It is possible that other substances and/or factors not listed above may interfere with the test and cause false positive results.*

**Endogenous and Preservative Compound Interference Study:** The following endogenous compounds were spiked into negative urine and the two levels of controls (375 ng/mL and 625 ng/mL) for the assay. The spiked solution was evaluated against the assay's calibration curve. Interference was observed with Ascorbic Acid, Boric Acid, and Citric Acid. No other major interference with these compounds at physiological relevant concentrations as all spiked samples gave correct corresponding preliminary positive/negative results against the cutoff value of 500 ng/mL. Results are summarized in the following table:

Endogenous Substance	Spiked [ ] (mg/dL)	Spiked Ethyl Glucuronide Concentration (ng/mL)		
		0 ng/mL	375 ng/mL Control	625 ng/mL Control
Acetone	1000	Neg	Neg	Pos
Ascorbic Acid	1500	Neg	Neg	Neg
Bilirubin	2	Neg	Neg	Pos
Boric Acid	1000	Neg	Neg	Neg
Calcium Chloride (CaCl <sub>2</sub> )	300	Neg	Neg	Pos
Citric Acid (pH 3)	800	Neg	Neg	Neg
Creatinine	500	Neg	Neg	Pos
Ethanol	1000	Neg	Neg	Pos
Galactose	10	Neg	Neg	Pos
γ-Globulin	500	Neg	Neg	Pos
Glucose	3000	Neg	Neg	Pos
Hemoglobin	300	Neg	Neg	Pos
β-hydroxybutyric Acid	100	Neg	Neg	Pos
Human Serum Albumin	500	Neg	Neg	Pos
Oxalic Acid	100	Neg	Neg	Pos
Potassium Chloride	6000	Neg	Neg	Pos
Riboflavin	7.5	Neg	Neg	Pos
Urea	6000	Neg	Neg	Pos
Uric Acid	10	Neg	Neg	Pos
Sodium Azide	1000	Neg	Neg	Pos
Sodium Chloride	6000	Neg	Neg	Pos
Sodium Fluoride	1000	Neg	Neg	Pos
Sodium Phosphate	300	Neg	Neg	Pos

The following endogenous compounds which showed interference at ±25 % of cutoff concentrations were then spiked into negative urine and at ±50 % of cutoff concentrations (250 ng/mL and 750 ng/mL) for the assay. Interference was still observed with Boric Acid and Citric Acid (pH 3). Results are summarized in the following table:

Endogenous Substance	Spiked [ ] (mg/dL)	Spiked Ethyl Glucuronide Concentration (ng/mL)		
		0 ng/mL	250 ng/mL Control	750 ng/mL Control
Ascorbic Acid	1500	Neg	Neg	Pos
Boric Acid	1000	Neg	Neg	Neg
Citric Acid (pH 3)	800	Neg	Neg	Neg

**pH Interference Study:** The pH levels for pooled negative human urine were adjusted to the following pH levels and spiked with analyte to the two levels of controls (375 ng/mL and 625 ng/mL) and tested by the assay. The pH adjusted solutions were evaluated against the assay's calibration curve. No major interference with these pH levels was observed as all pH adjusted levels gave correct corresponding preliminary positive/negative results against the cutoff value of 500 ng/mL. Results are summarized in the following table:

pH	Spiked Ethyl Glucuronide Concentration (ng/mL)		
	0 ng/mL	375 ng/mL Control	625 ng/mL Control
pH 3	Neg	Neg	Pos
pH 4	Neg	Neg	Pos
pH 5	Neg	Neg	Pos
pH 6	Neg	Neg	Pos
pH 7	Neg	Neg	Pos
pH 8	Neg	Neg	Pos
pH 9	Neg	Neg	Pos
pH 10	Neg	Neg	Pos
pH 11	Neg	Neg	Pos

**Specific Gravity:** Samples ranging in specific gravity from 1.000 to 1.030 were split into three portions each and either left un-spiked or further spiked to a final ethyl glucuronide concentration of either 375 ng/mL or 625 ng/mL (the negative and positive control concentrations, respectively). These samples were then evaluated in semi-quantitative and qualitative modes. No interference was observed.

## Bibliography:

1. Urine Testing for Drug of Abuse, National Institute on Drug Abuse (NIDA) Research Monograph 73, 1986.
2. Mandatory Guidelines for Federal Workplace Drug Testing Program, National Institute on Drug Abuse, Federal Register, **53** (69):11970 (1988).
3. Schmitt G., Aderjan R., Keller, T., Wu, M., Ethyl glucuronide: an unusual ethanol metabolite in humans. Synthesis, analytical data, and determination in serum and urine, *J. Anal. Toxicol.*, **19**(2):91-4 (1995).
4. Dahl, H., Stephanson, N., Beck, O., and Helander, A., Comparison of urinary excretion characteristics of ethanol and ethyl glucuronide, *J Anal Toxicol*, **26**(4):201-4 (2002).
5. Helander, A., and Beck, O., Ethyl sulfate: a metabolite of ethanol in humans and a potential biomarker of acute alcohol intake, *J Anal Toxicol.*, **29**(5):270-4 (2005).
6. Bergström, J., Helander, A., and Jones, A.W., Ethyl glucuronide concentrations in two successive urinary voids from drinking drivers: relationship to creatinine content and blood and urine ethanol concentrations, *Forensic Sci Intl.*, **133**(1-2):86-94 (2003).
7. Wurst, F.M., and Metzger, J., The ethanol conjugate ethyl glucuronide is a useful marker of recent alcohol consumption, *Alcohol Clin Exp Res.*, **26**(7):1114-9 (2002).
8. Jones, A.W., Urine as a biological specimen for forensic analysis of alcohol and variability in the urine-to-blood relationship, *Toxicol Rev.*, **25**(1):15-35 (2006).
9. Høiseth, G., Bernard, J.P., Karinen, R., Johnsen, L., Helander, A., Christophersen, A.S., and Mørland, J., A pharmacokinetic study of ethyl glucuronide in blood and urine: applications to forensic toxicology, *Forensic Sci Intl.*, **172**(2-3):119-24 (2007).
10. Helander, A., Beck, O., and Jones, A.W., Laboratory testing for recent alcohol consumption: comparison of ethanol, methanol, and 5-hydroxytryptophol, *Clin. Chem.*, **42**(4):618-24 (1996).
11. Schmitt, G., Droenner, P., Skopp, G., and Aderjan, R, Ethyl glucuronide concentration in serum of human volunteers, teetotalers, and suspected drinking drivers, *J. Forensic Sci.* **42**(6):1099-102 (1997).
12. Borucki, K., et al., Detection of recent ethanol intake with new markers: comparison of fatty acid ethyl esters in serum and of ethyl glucuronide and the ratio of 5-hydroxytryptophol to 5-hydroxyindole acetic acid in urine, *Alcohol Clin Exp Res.* **29**(5):781-7 (2005).
13. Stephanson, N., Dahl, H., Helander, A., and Beck, O., Direct quantification of ethylglucuronide in clinical urine samples by liquid chromatography-mass spectrometry, *Ther Drug Monit*, **24**(5):645-51 (2002).
14. Seidi, S., et al., Ethyl glucuronide – A Biological Marker for recent alcohol consumption, *Addiction Biology*, **6**(3):205-212 (2001).
15. Kipper, G.E., et al., Ethyl glucuronide: A Biomarker to identify Alcohol use by Health Professionals Recovering from Substance use Disorders, *Alcohol and Alcoholism*, **39**(5): 445-449 (2004).
16. Wurst, F.M., et al., Ethyl glucuronide – A marker of Recent Alcohol Consumption with Clinical and Forensic Implications, *Alcohol*, **20**(2):111-116 (2000).
17. Skipper, G.E., et al., Ethyl glucuronide (EtG): A new marker to detect Alcohol use in recovering physicians, *Journal of Medical Licensure and Discipline*, **90**(2):14-17 (2004).
18. Wurst, F.M., et al., Ethyl glucuronide Discloses Recent Covert Alcohol Use Not Detected by Standard Testing in Forensic Psychiatric Inpatients, *Alcoholism: Clinical and Experimental Research*, **20**(3):471-476 (2003).
19. Rubenstein, K.E., Schneider, R.S., and Ullman, E.F., Homogeneous Enzyme Immunoassay: A New Immunochemical Technique, *Biochem Biophys Res Commun*, **47**:46 (1972).
20. Sodium Azide. National Institute for Occupational Safety (NIOSH). Pocket Guide to Chemical Hazards. Third Printing, September 2007. Available online at: <https://www.cdc.gov/niosh/npg/default.html>
21. Schloegl, H., et al., Stability of ethyl glucuronide in urine, post-mortem tissue and blood samples, *International journal of legal medicine*, **120**(2):83-88 (2006).
22. Nichols, J., Instrument Validation: The Road to Success. CLN's Lab 2004: From Basic to Advanced Series. 14-16 (2004).
23. CDRH Guidance for Industry and FDA Staff: Replacement Reagent and Instrument Family Policy (2003).

Additions, deletions, or changes are indicated by a change bar in the margin.

For technical assistance please call: (408) 970-8811

**Manufacturer:**  
**Lin-Zhi International, Inc.**  
2945 Oakmead Village Court  
Santa Clara, CA 95051  
USA  
Tel: (408) 970-8811  
Fax: (408) 970-9030  
[www.lin-zhi.com](http://www.lin-zhi.com)



**Authorized European  
Rep. within the EU:**  
CEpartner4U  
Esdoornlaan 13  
3951 DB Maarn  
The Netherlands  
[www.cepartner4u.eu](http://www.cepartner4u.eu)



© November 2019 Rev. 0