

Please note that information in this cross-reactivity profile contains information compiled from LZI product inserts as well as compounds tested following FDA 510k submission and therefore provides data that has NOT been FDA 510k cleared. This data is not intended to modify nor can it be used to change the original intended use of the assay. Original product inserts should be referred to for the most updated and approved information on specificity and cross-reactivity.

Regarding the "Cross": The concentration listed under "Concentration Tested" is the concentration that most closely approximates the cutoff concentration. Concentrations above the listed amount may elicit a false-positive from this assay.

* Please refer to the "Instructions" page for information regarding the "concentration tested".

Assay 6-AM
Catalog # 0290, 0291
Cutoff 10 ng/mL

Generic/Chemical Name	Brand Name	Cross	Concentration Tested (ng/mL)
Codeine		N	500,000
Dextromethorphan		N	100,000
Dihydrocodeine		N	500,000
Heroin		Y	10
Hydrocodone		N	300,000
Hydromorphone		N	100,000
Imipramine		N	200,000
Levorphanol		N	100,000
Meperidine		N	800,000
Morphine		N	100,000
M3G		N	600,000
M6G		N	600,000
Nalorphine		N	100,000
Naloxone		N	500,000
Naltrexone		N	300,000
Norcodeine		N	600,000
Normorphine		N	100,000
Oxycodone		N	500,000
Oxymorphone		N	100,000

11-nor- -THC-COOH		N	100,000
Acetaminophen		N	500,000
Acetylsalicylic Acid		N	500,000
Amitriptyline		N	500,000
Benzoylcegonine		N	500,000
Brompheniramine		N	100,000
Caffeine		N	500,000
Chlorpomazine		N	250,000

Please note that information in this cross-reactivity profile contains information compiled from LZI product inserts as well as compounds tested following FDA 510k submission and therefore provides data that has NOT been FDA 510k cleared. This data is not intended to modify nor can it be used to change the original intended use of the assay. Original product inserts should be referred to for the most updated and approved information on specificity and cross-reactivity.

Regarding the "Cross": The concentration listed under "Concentration Tested" is the concentration that most closely approximates the cutoff concentration. Concentrations above the listed amount may elicit a false-positive from this assay.

* Please refer to the "Instructions" page for information regarding the "concentration tested".

Assay 6-AM
Catalog # 0290, 0291
Cutoff 10 ng/mL

Generic/Chemical Name	Brand Name	Cross	Concentration Tested (ng/mL)
Desipramine		N	500,000
Diazepam	Valium	N	100,000
Digoxin		N	100,000
Diphenhydramine		N	100,000
Doxepin		N	100,000
Fluoxetine		N	500,000
Hydroxyzine		N	500,000
Ibuprofen		N	500,000
Methadone		N	500,000
Methamphetamine		N	500,000
Oxazepam		N	500,000
Phencyclidine		N	100,000
Phenobarbital		N	500,000
Propoxyphene		N	100,000
Ranitidine		N	500,000
Secobarbital		N	500,000
Triprolidine		N	100,000

Soma	Carisoprodol	N	1,000 µg/mL
Sertraline	Zoloft		NSS
25I-NBOMe		N	200,000 ng/mL
Acetaminophen	Nyquil	N	NSS
Aripiprazole	Abilify	N	NSS
Adalimumab	Humira	N	NSS
Atorvastatin	Lipitor	N	NSS
Benzocaine	Amesol, Cepacol, Lanacane	N	NSS
Bevacizumab	Avastin	N	NSS
Bismuth subsalicylate	Pepto-Bismol	N	NSS

Please note that information in this cross-reactivity profile contains information compiled from LZI product inserts as well as compounds tested following FDA 510k submission and therefore provides data that has NOT been FDA 510k cleared. This data is not intended to modify nor can it be used to change the original intended use of the assay. Original product inserts should be referred to for the most updated and approved information on specificity and cross-reactivity.

Regarding the "Cross": The concentration listed under "Concentration Tested" is the concentration that most closely approximates the cutoff concentration. Concentrations above the listed amount may elicit a false-positive from this assay.

* Please refer to the "Instructions" page for information regarding the "concentration tested".

Assay 6-AM
Catalog # 0290, 0291
Cutoff 10 ng/mL

Generic/Chemical Name	Brand Name	Cross	Concentration Tested (ng/mL)
Calcium carbonate	Rolaid	N	NSS
Calcium carbonate	Tums	N	NSS
Cetirizine	Zyrtec-D	N	NSS
Chlorphenamine	Allerest	N	NSS
Chlorphenamine	Chlor-Trimeton	N	NSS
Chlorpheniramine	Actifed	N	NSS
Chlorpheniramine	Coricidin	N	NSS
Cimetidine	Tagamet HB	N	NSS
Clemastine	Tavist-1	N	NSS
Clopidogrel	Plavix	N	NSS
Codeine	Robitussin	N	NSS
Cromoglicic acid	Nasalcrom	N	NSS
Dextromethorphan	Coricidin	N	NSS
Dextromethorphan	Nyquil	N	NSS
Doxylamine succinate	Nyquil	N	NSS
Duloxetine	Cymbalta	N	NSS
Etanercept	Enbrel	N	NSS
Fluticasone	Advair	N	NSS
Guaifenesin	Mucinex	N	NSS
Guaifenesin	Robitussin	N	NSS
Imatinib	Gleevec	N	NSS
Infliximab	Remicade	N	NSS
Insulin glargine	Lantus	N	NSS
Loperamide	Imodium	N	NSS
Loratadine	Alavert, Claritin	N	NSS
Magnesium hydroxide	Rolaid	N	NSS
Naproxen	Aleve	N	NSS
Nizatidine	Axid AR	N	NSS
Omeprazole	Prilosec	N	NSS
Oxymetazoline	Afrin	N	NSS

Please note that information in this cross-reactivity profile contains information compiled from LZI product inserts as well as compounds tested following FDA 510k submission and therefore provides data that has NOT been FDA 510k cleared. This data is not intended to modify nor can it be used to change the original intended use of the assay. Original product inserts should be referred to for the most updated and approved information on specificity and cross-reactivity.

Regarding the "Cross": The concentration listed under "Concentration Tested" is the concentration that most closely approximates the cutoff concentration. Concentrations above the listed amount may elicit a false-positive from this assay.

* Please refer to the "Instructions" page for information regarding the "concentration tested".

Assay 6-AM
Catalog # 0290, 0291
Cutoff 10 ng/mL

Generic/Chemical Name	Brand Name	Cross	Concentration Tested (ng/mL)
Phenol	Chloraseptic	N	NSS
Phenylephrine	Actifed	N	NSS
Phenylephrine	Contac	N	NSS
Phenylephrine	Neo-Synephrine	N	NSS
Phenylephrine	Thera-flu	N	NSS
Phenylpropanolamine	Allerest	N	NSS
Pseudoephedrine	Afrinol, Sudafed	N	NSS
Pseudoephedrine	Robitussin	N	NSS
Pseudoephedrine	Sudafed	N	NSS
Pseudoephedrine	Zyrtec-D	N	NSS
Rituximab	Rituxan	N	NSS
Rosuvastatin	Crestor	N	NSS
Salmeterol	Advair	N	NSS
Simethicone	Mylicon	N	NSS
Tiotropium bromide	Spiriva	N	NSS
Trastuzumab	Herceptin	N	NSS
Quinapril	Accupril	N	NSS
Clonazepam	Klonopin	N	NSS
Duloxetine	Cymbalta	N	NSS
Gabapentin	Neurotin	N	NSS
Norvasc	Amlodipine	N	NSS
Omeprazole	Prilosec	N	NSS
Protonix	Pantoprazole	N	NSS
Zolpidem	Ambien	N	NSS
	Metformin	N	NSS
	Provastatin	N	NSS
Pentazocine	Talwin	N	100,000 ng/mL
6-Acetylcodeine		N	100,000 ng/mL